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## Chaise portoir

Réf. SBRAN15\_O

### I. Descriptif :

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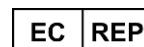
Zhangjiagang Xiehe Medical Apparatus & Instruments Co.,Ltd.  
 NO.7th, MiddleXinzha Road, Zhashang Industrial Zone,  
 Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.  
 Fascinatio Boulevard 522, Unit 1.7,  
 2909VA Capelle aan den IJssel  
 Netherlands

# User Manual Stair Stretcher

Réf. SBRAN15\_O

## I. Technical Parameters:

Product Size (L×W×H): 84×60×98cm

Folding Size (L×W×H): 98×60×20cm

Packing Size (1pc/carton): 100×61×22cm

N.W.: 14kg G.W.: 15.5kg

Loading Bearing: less than 159kg

## II. How to use:

This staircase stretcher with its high-strength aluminum alloy frame is mainly used for transporting patients up and down stairs. It enables the patient to be placed quickly and gently on the stretcher, operating table or hospital bed. It can then be removed from under the patient.

The two rear wheels have brakes and are fixed, while the two front wheels swivel and have no brakes. This means it can be moved over most surfaces with minimum effort.

Patient safety during the transport process is ensured by 2 restraint

straps with quick-release buckles, armrests and two retractable arms at the front of the stretcher.

## III. Cautions:

When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety's sake. In general operation, be careful not to out torn the stretcher surface.

## IV. Maintenance:

Keep clean as routine (incl. sterilization). Clean with a surface disinfectant.

Often check if any parts loose or not.

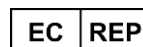
## V. Storage & transportation:

Store in a place protected from humidity, which could lead to product corrosion.

	Manufacturer
<b>EC REP</b>	Authorised representant of the European Union
	Importer
	Distributor
<b>REF</b>	Holtex reference
<b>LOT</b>	Lot number
	Manufacturing date
<b>MD</b>	Medical device

Zhangjiagang Xiehe Medical Apparatus & Instruments Co.,Ltd.  
NO.7th, MiddleXinzha Road, Zhashang Industrial Zone,  
Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.  
Fascinatio Boulevard 522, Unit 1.7,  
2909VA Capelle aan den IJssel  
Netherlands