# Combat Application Tourniquet

**Notice:** When applied in accordance with direction, this tourniquet is a safe and effective device for controlling life-threatening extremity bleeding. If you cannot be sure or cannot take the additional time to examine where the bleeding is coming from based on the situation, the tourniquet can be effectively applied over clothing as high on the arm or leg as possible.

The tourniquet **must not be applied over solid objects within the clothing**. **As soon as the situation permits**, the injured limb should be evaluated and the tourniquet re-positioned 2-3 inch above the injury directly to the skin.

1. Apply tourniquet proximal to the bleeding site and place the wounded limb through the band loop.

2. Pull the band very tight and fasten it back on itself

4. Twist the rod until bright red bleeding stops.

3. Adhere the band around the limb.

5. Place the rod inside the clip and lock.

6. Adhere the band over the rod, for small extremities, continue to adhere the band around the extremity and over the windlass rod.

7. Secure the rod and band with strap, adhere it to the opposite hook on the windlass clip. Record time of application.

## Warning:

Please read entire instruction for use prior to using this product. Inappropriate application can result in serious injury or loss of limb.

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**Shenzhen TMI Medical Supplies Co., Ltd**
**SUNGO Europe B.V.**