MIROIR FRONTAL DE CLAR

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CLAR HEAD MIRROR

Thank you for purchasing this product.

ATTENTION :

The ENT forehead mirrors are local lighting devices with luminous forehead mirror particularly useful to examine the oral cavity, the nose and the ear. These products are equipped with mirror made of special enameled glass and are assembled on a practical helmet.

FEATURES

The packaging includes Clar Head Mirror having 2 pre-loaded CR123 batteries in the battery box.

This product shall be used by qualified personnel only.

PRESCRIPTIONS

The Mirrors are very delicate and shall be handled carefully. Broken Bulbs or mirrors can cause injuries or cut. Carefully handle and immediately replace.

Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions.

Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

INSTALLATION

The device is available with 3.0V led bulb or 6V Bulb

Once the system is setup up and light is turned on, the diameter/ Size of the spot can be adjusted by moving the bulb towards or away from the mirror. The Position of the Spot can be adjusted by using coupler screw.

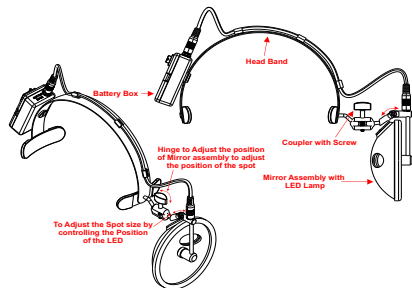
MAINTENANCE

The product has been conceived to last forever. It is resistant to most of chemicals except for methyl ethyl ketones.

Do not use abrasive materials to clean the mirror, which could cause permanent damages.
Before removing the bulb, make sure that the instrument has been turned off for some minutes to avoid the risk of burning of figures.
- Cold clean the helmet with some disinfectant and sponge.
- Clean the mirror with an adequate detergent and cotton cloth.

INSTALLATION INSTRUCTION WITH HEAD BAND MOUNTED BATTERY BOX

Note:

These Instructions are applicable for both 55mm and 100mm Clar Head Mirrors.

WARRANTY

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control.

The product has a 24-month warranty, Valid from the date it is delivered by the manufacturer.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labor, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as part in rubber or plastic, lamps batteries, electrodes, washers and packing etc. No component will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

This guarantee is void in the event of:

- repairs effected by unauthorized persons or with spare parts and approved by the manufacturer.
 - Damages or defects caused by negligence blows.
 - Abnormal use of the appliance or faulty installation.
- Faulty appliances must be returned only to the retailer where the items was bought. Any item dispatched directly to us will be refused.*

CAUTION

Attention: See instructions for use.

Spengler

The Product is ISO Certified Company

Date de 1^{re} marquage CE : 30-09-2005
1^{re} CE marking date: 2005-09-30

SPENGLER SAS
Address: 30 rue Jean de Guiramaud,
13290 Aix-en-Provence, FRANCE

Medical Devices (Pvt) Ltd.
Wazirabad Road, Ugoki, Sialkot - Pakistan
Tel: 0092-52-3571214, 3571215, Fax: 0092-52-3571210, 3571211, Web: www.mdevices.com
Email: info@mdevices.com

EC Representative OBELIS s.a
Bd.Général Wahis 5310, O Brussels, BELGIUM

Spengler

MIROIR FRONTAL DE CLAR CLAR HEAD MIRROR

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NULLMIROIRS_IVL_100822

WWW.SPENGLER.FR