Pied à sérum - 2 & 4 crochets
Drip stand - 2 & 4 hooks
شماعة ملابس - 2 و 4 خطاف

Sommaire / Contents / الفهرس

Pièces détachées / spare parts / قطع الغيار	2
Etapes de montage / assembly steps / مراحل التركيب	3
Garantie / guarantee / الضمان	5

Importé et distribué par / Imported and Distributed by مستورد وموزع بمعرفة

Holtex

30 rue Jean de Guirmand
13290 Aix-en-Provence
France

+33 (0)4 42 90 31 31

info@holtex.fr

www.holtex.fr

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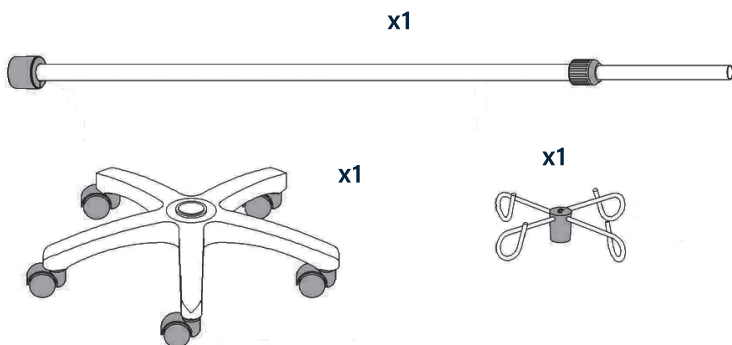
Danyang New Hope Medical Equipment Ltd.

Address: No 889, Zhongxin West Rd,
Danbei Town, danyang City, China
www.medicalcn.com

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For after sales service, please download the form
from our website at :
www.gsh-med.fr

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Insert the stainless steel rod into the caster base.

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
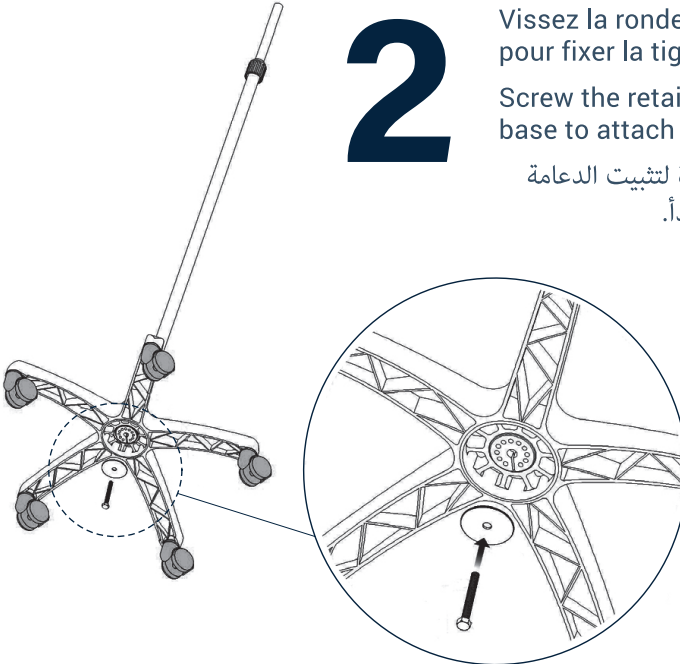
Screw the retaining nut under the caster base to attach the stainless steel rod.

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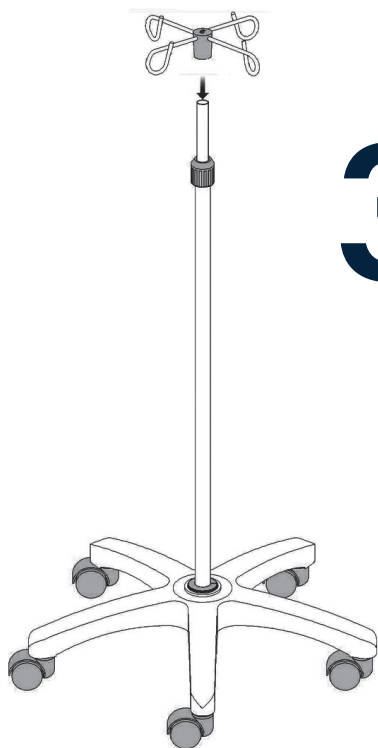Insert the hooks into the end of the stainless steel rod.

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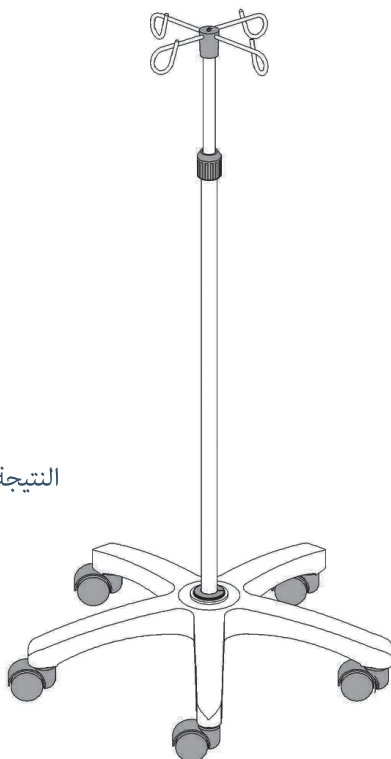
EN Thank you for purchasing a Holtex product. Holtex furniture is guaranteed for one year and has been rigorously quality checked so that we can guarantee its replacement or repair should any proven defect appear during the guarantee period.

Holtex promises to replace defective parts returned (after a check is made by Holtex after-sales service) during the applicable guarantee period and as part of normal product use.

The guarantee does not cover:

- Wearing parts and consumables
- Travel and labour costs
- Failures that result from improper use, obvious negligence or transporting equipment
- Damage, breakdowns, failures or faults due to outside causes (natural disasters, electric power overvoltages, impacts, presence of foreign objects, etc.)
- Damage due to corrosion or gradual product deterioration
- Scratching, chipping or scuffing of the equipment that does not affect its proper operation
- Equipment where the serial number has been modified, made illegible or removed.

Repair or replacement under guarantee does not extend or renew the guarantee period.

The guarantee only applies if the complete product is returned along with the bill or original sales receipt issued in the name of the consumer by the retailer, who must have filled in the After-Sales form available for download from our website at www.gsh-med.fr

Holtex will ensure the availability of spare parts for products that it no longer sells for a period of two years after the end of the guarantee period.

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