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Civière de Relevage Aluminium

Réf. SBRAN12

I. Descriptif :

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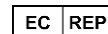
ZHANGJIAGANG XIEHE Medical Apparatus & Instruments Co.,Ltd.
 NO.7th, Middle Xinzha Road, Sigang Village,
 Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.
 Fascinatio Boulevard 522, Unit 1.7,
 2909VA Capelle aan den IJssel
 The Netherlands

User Manual Scoop Stretcher Réf. SBRAN12

I. Technical parameters:

Product size (L x W x H): 210 x 44 x 9 cm
 Folding size: 120 x 44 x 6 cm
 Packing size: 170 x 45 x 9cm
 Net size: 8.5kg/ Growth size: 10.2kg
 Load bearing: less than 159kg

II. How to use:

It is a separately-type stretcher to transfer fractured and serious patients. At both ends of the stretcher, a hinged and joined arrangement is set in the middle, which can set the stretcher into two parts, and it is easy to lock and unlock. The stretcher can strengthen the patients at original place; it can reduce the further hurt of the patients, the patients can be quickly & easily put in the stretcher and also the stretcher can be taken out from the back of the patient without moving the patient. The length of the scoop stretcher may be adjusted to match the patient's body length. It can then easily load a patient onto an ambulance stretcher and immediately separate into two sections for folding.

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

III. Cautions:

When carrying patients with this stretcher, be sure to lock tightly the fender locking device & fasten the safety belt for safety sake. In general operation, be careful not to torn out the stretcher surface.

IV. Maintenance:

Keep clean as routine (incl. sterilization).
 Often check if any parts loose or not.


V. Storage & transportation:

Store this product in the place of damp proof & non-corrosion.
 Common transporting vehicle can carry this product.

	Manufacturer
EC REP	Authorised representant of the European Union
	Importer
	Distributor
REF	Holtex reference
LOT	Lot number
	Manufacturing date
MD	Medical device

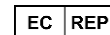ZHANGJIAGANG XIEHE Medical Apparatus & Instruments Co.,Ltd.
 NO.7th, Middle Xinzha Road, Sigang Village,
 Yangshe Town, Zhangjiagang City, Jiangsu Province 215600, China

SUNGO EUROPE B.V.
 Fascinatio Boulevard 522, Unit 1.7,
 2909VA Capelle aan den IJssel
 The Netherlands