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The ASKIR 118 SUPPORT bracket (REF ASU 118) has been designed and manufactured to allow the NEW ASKIR 118 BASIC medical suction unit to be fixed to the wall of medical and emergency vehicles.

The support bracket is equipped with an electrical cable, which is connected to an external DC source (12V $\overline{---}$), that allows the internal battery of the suction unit to be charged.

As it is easy to fix and is compliant with the standards in force, its protection in case of collisions, accidents and/or overturning of medical devices is guaranteed. The ASKIR 118 SUPPORT bracket is connected to the cable of the 12V $\overline{---}$ power supply source and allows the internal battery of the suction unit to be charged.

WARNINGS AND PRECAUTIONS

- Carefully read this user manual before fitting, fixing, and connecting the ASKIR 118 SUPPORT bracket (REF ASU 118);
- The bracket must be installed in compliance with the indications provided in this manual. Failure to comply with such warning may compromise the conformity to standard EN 1789 (design and construction of medical vehicles) and significantly reduce the safety of the medical device;
- Never alter mechanical, electrical, and structural parts of the bracket. These interventions make the device dangerous and, therefore, they preclude its use;
- No technical intervention on the device must be carried out by the user. The only operations allowed are those indicated in this manual. For any technical problem, revisions and repairs, refer to the authorised support service and/or the manufacturer.
- Always use spare parts supplied by CA-MI in order to guarantee maximum efficiency and safety of the device;
- Do not remove and/or modify the retaining spring in order to ensure the seal in the event of negative acceleration forces (max. 10 G);

The electrical connections must always be performed by qualified and authorised personnel. Connections must never be entrusted to unauthorised personnel.

Non-compliant connections to electrical systems and/or connections made by unqualified installers may damage the electrical system.

Only qualified personnel are authorised to replace the ASU 118 power cord. In the event that the service personnel has to replace the power cord is recommended that the correct connection and proper fastening of the same component.

CONTROLS, MECHANICAL DRIVES, ELECTRICAL CONNECTIONS

The support bracket of the NEW ASKIR 118 BASIC device consists of a plastic part (ABS) and a seal hook, which allows the suction unit to be fixed safely and easily. The figure below shows the bracket complete in all its parts.

- The bracket must be installed in compliance with the indications provided in this manual. Failure to comply with such warning may compromise the conformity to standard EN 1789 (design and construction of medical vehicles) and significantly reduce the safety of the medical device;
- Never alter mechanical, electrical, and structural parts of the bracket. These interventions make the device dangerous and, therefore, they preclude its use;

This bracket is equipped with electrical cable connected to the 12V $\overline{---}$ power supply source of the ambulance and allows the internal battery of the suction unit to be charged.


The charge system (contact supports) has been made so as to make short circuiting difficult between the two contacts (positive and negative). The electrical cable of the bracket has double insulation and it must always be connected to an electrical source (12V $\overline{---}$) of the ambulance. Connection to the power supply source must always comply with the polarity. In case of incorrect connection, the internal battery cannot be charged.

The support bracket has been manufactured to withstand the weight of the NEW ASKIR 118 BASIC suction unit. Statistical calculations allowed the dimensioning of the hooks and retaining spring for negative accelerations (referred to the force of gravity) up to 10G.

INSTALLING THE BRACKET TO THE WALL

Before installing the bracket, make sure that the space available is sufficient to house also the suction unit (for overall dimensions, refer to figure 2). Installation in emergency vehicles must be made at a maximum height of 130 cm to allow the device to be easily hooked and unhooked (consider also the weight of the device, which is equal to approximately 2.7 Kg).

MAXIMUM OVERALL DIMENSIONS OF THE NEW ASKIR 118 BASIC DEVICE WITH BRACKET (FIG.1)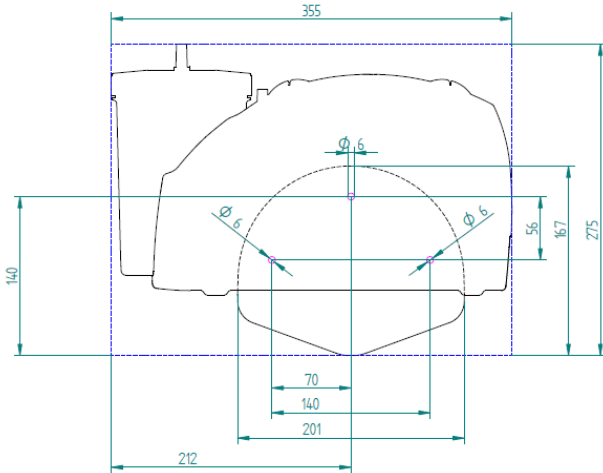

- Trace the holes centres on the mounting wall (see Fig.2);


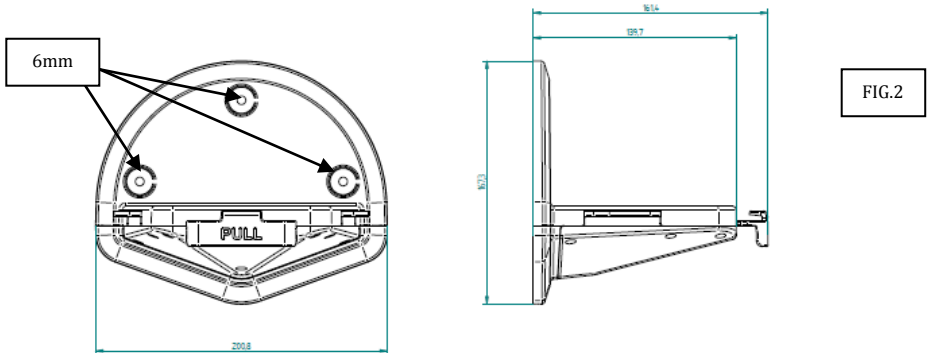
FIG.2

- Drill a hole to fix the power supply cable with a diameter of at least 8 mm. Before inserting the power supply cable you can insert a rubber cable gland with a diameter of at least 6 mm to prevent the cable insulation sheath from damaging;
- Fasten the three TCEI 6x30 screws together with the 6x18 washers, whereas, on the side of the plate, everything must be fastened with a 6x24 flat washer, diam. 6 toothed washer, and M6 nut.
- Figure 3 shows the correct installation of the bracket support

- The fixing bracket cannot be fixed to one single external wall cladding. The organiser of the medical vehicle must provide a suitable metal reinforcement complete with threaded nuts (to fix the bracket) under the wall cladding. This reinforcement must be suitably designed to guarantee perfect seal in case of negative accelerations up to 10 g, according to that described in standard EN 1789.
- Do not enlarge and/or deform the holes of the support bracket;
- Do not alter and/or deform the safety spring and/or electrical contacts;

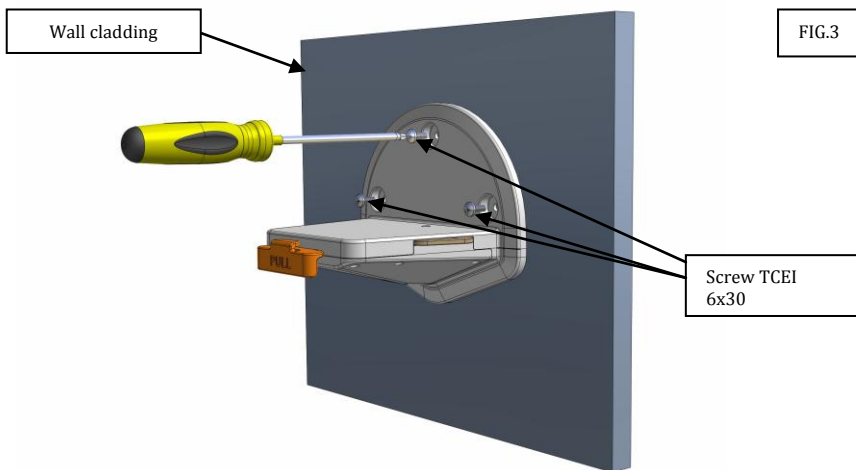
Do not force the bracket to fit the screws (e.g. due to incorrect or moved holes)

CHARGE CABLE CONNECTION


Once the bracket has been fixed to the wall and its operation has been checked, connect the electrical cable to the 12V  power supply source.

The connection of the bracket to the power supply source must be made by complying with the polarity (indicated on the cable). In case of incorrect connection, the internal battery of the suction unit cannot be charged.

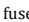Although the NEW ASKIR 118 BASIC (REF RE 410170) device is protected by a **T 15A L 125V** fuse, which protects the device itself from possible short circuits and malfunctions on the bracket and/or the connected aspirator, the installer must make sure that the 12V  power source is protected by a fuse of at least 15A 125V (with delayed function).

The electrical connections must always be performed by qualified and authorised personnel. Connections must never be entrusted to unauthorised personnel.

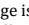Even if voltage is only 12V , short circuit may cause serious damage to persons and objects (risk of fire, etc.). It is the installer's responsibility to provide a means of disconnection from the ambulance electrical source that must be installed externally to the ASU118 bracket (designed to allow for wall installation of the NEW ASKIR 118 BASIC (REF RE 410170) medical aspirator in first-aid and emergency vehicles).

ANCHORAGE SURGICAL ASPIRATOR NEW ASKIR 118 BASIC

The ambulance adapter has been designed and built exclusively for attachment of the vacuum and recharge NEW ASKIR 118 BASIC and therefore no other type of device can be anchored in such a system.

The sealing efficiency and charging the device can only be guaranteed for this model of surgical aspirators .

Operations of insertion and extraction of the surgical aspirator:

- Take the vacuum cleaner NEW ASKIR 118 BASIC;
- Place the device on undone support and pulling the hook sealing try to make the same fit within the section on the bottom of the device;
- Before removing your hands from the extractor to ensure the correct fitting (by pulling the handle of the device upwards verifying the correct placement of the device on the bracket);

- To remove the extractor from the bracket, pull the hook seal making it leak from the section on the bottom of the device. Hold the device securely and remove it from the bracket by placing it in a safe manner.

FIG.4

At the end of the insertion of the vacuum in the ambulance adapter to ensure charging is in progress by checking the indicator on the LCD screen on the front of the device as TAB.II

SUPPORT BRACKET FUNCTIONAL TEST

The test operations described herewith allow the user to check the efficiency of the support, the correct charge of the suction unit and/or to assess whether the intervention of the technical service is required. This check must be performed at least once a day and, anyway, always once a week.

- Check the operation of the bracket (without suction unit) by repeatedly acting on the seal hook. The movement must be free from obstructions.
- Always make sure that the fixing screws are tightened correctly;
- Fit the suction unit in the bracket as indicated in Chapter "Securing the NEW ASKIR 118 BASIC suction device";
- Visually check (as per TAB I and II) the activation of the LED.

In the event one or more phases fail, refer to the technical service.
Do not tamper with mechanical and/or electrical parts as this may affect the safety and efficiency of the device.
No electrical and/or mechanical part of the support bracket has been designed to be repaired by the manufacturer, customer, and/or user. Always refer to the authorised technical service.

TAB. I - LIGHT INDICATIONS DURING OPERATION

Back-light signals	Phase	Problem / Cause	Solution
Flashing Green Led	During recharge	Battery charge in progress	Wait
Steady Greed Led	During recharge	Charging cycle complete	Remove the device from the support plate (ASKIR 118 SUPPORT mod.ASU 118) of the ambulance

TAB. II - SIGNALS INDICATOR / LED INDICATOR BUTTON

Button LED signal / written signal	Function	Colour	Position
ON/OFF button	Power on	Green / Yellow / Red	Near the front panel key
Proximity Button	Switch the ON/OFF key	Blue	LED located above the "Proximity" key

MAINTENANCE AND REUSE

Once the device has been fitted in the bracket, always make sure that, upon activation, the LED (ref. TAB.I e TAB.II) confirms the charge in progress of the internal battery. When using emergency vehicles, always check the housing of the suction unit in the support bracket at the end of the intervention. In case of accidents or collisions of the emergency vehicle, always request the authorised technical service to check the support bracket and the suction unit.

DISMANTLING THE SUPPORT BRACKET

Follow the operations below to replace and/or dismantle the support bracket:

- Remove the device from the support bracket;
- Disconnect the power supply cable (refer to authorised personnel);
- Loosen the screws of the bracket and store them in a safe place;
- Remove the power supply cable and store it in a safe place.

SYMBOLS

	General warnings and/or specifications
	Consult the instruction manual
	CE marking in conformity with EC directive 93/42/CEE and subsequent changes
	The device contains electrical and/or electronic parts which must be disposed of according to European Directive 2012/19/EU
	Direct Current
	Manufacturer : CA-MI Srl - Via Ugo La Malfa n°13 Frazione Pilastro, 43013 Langhirano (PR) Italy
	Batch production
	Model / Ref number

TECHNICAL CHARACTERISTICS

Size	200 x 167 (h) x 161 mm
Weight	514gr (including power supply cable)
Tolerances on all measurements	± 5%
Power Feeding	12V ===
Fuses	Inside the NEW ASKIR 118 BASIC device (F 15A L 125V)
Working Condition	Room temperature: 0 ÷ 40 °C Room humidity percentage: 0 ÷ 85% RH Atmospheric pressure: 800 ÷ 1060 hPa Altitude: 0 ÷ 2000m s.l.m.
Conservation condition and Transport	Room temperature (≤ 1 month): : - 20°C ÷ 45°C Temperatura ambiente (≤ 3 month): - 20°C ÷ 35°C Temperatura ambiente (≤ 1 year): 0°C ÷ 25°C Room humidity percentage: 0 ÷ 85% RH Atmospheric pressure: 500 ÷ 1060 hPa

IMPORTANT INFORMATION FOR CORRECT DISPOSAL OF THE PRODUCT IN ACCORDANCE WITH EC DIRECTIVE 2012/19/UE-RAEE: The symbol on the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, don't dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding protecting and improving environment quality, as well as avoiding potential effects on human

health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.
Caution: The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY CONDITIONS

This product is guaranteed for a period of **24 months** from the date of purchase. The warranty includes the repair or replacement of defect spare parts free of charge, if the defect has been clearly described by the customer and determined by technical service. Inspections on the part of the seller, performed at the request of the customer and intended to determine whether the device is fully functional, are not covered by the free-of-charge warranty service. This service will be charged to the customer depending on the effort required. The consumable components are not subject to warranty. Consumable components are silicon tubes, filters, seals, conical adaptor and suction catheter. Also excluded from warranty is all damage resulting from improper handling, wilful damage or improper care of the device.

The warranty shall expire if repairs and servicing are not carried out by technical service.

The manufacturer cannot be held liable for accidental or indirect damages should the device be modified, repaired without authorization or should any of its component be damaged due to accident or misuse. Any minimal modification / repair on the device voids the warranty and does not guarantee the compliance with the technical requirements provided by the MDD 93/42/EEC (and subsequent changes) and its normatives.

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